


Olberon Vacuderm™

Optimising venous access


Instructions for use

1

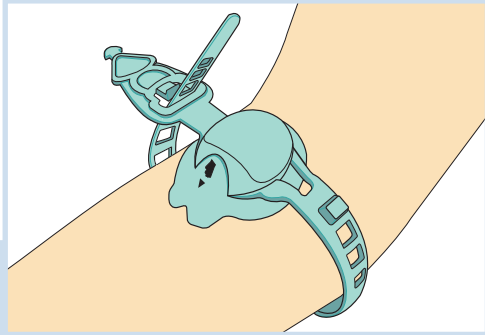


Ensure the hand/arrow V section is pointing towards the patient's hand to allow for accurate alignment to the vein.

4

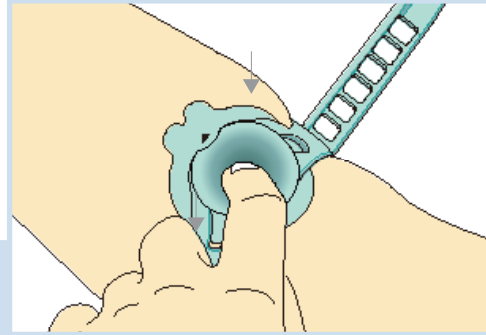


Pump the dome at least 10 to 15 times, **gently and slowly** towards the back of the dome to optimise vein distension. Continue pumping a few more times if the desired effect has not yet been reached. The vein should feel soft and spring back ready for cannulation.

2

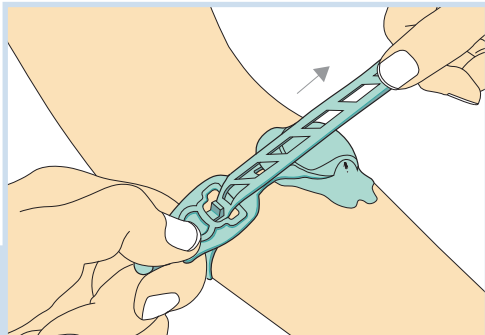


Using two hands, thread the strap through the buckle and stretch, ensuring the tourniquet is pulled as **tightly** as it will allow to give the required tourniquet result.

5

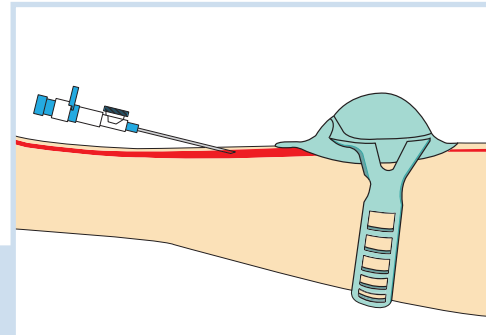


Insert the cannula as per standard practice.

3

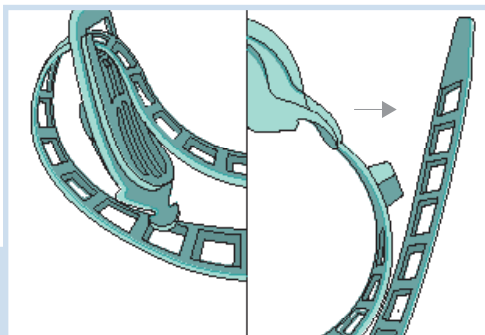

The excess strap can be hooked back on itself.

6

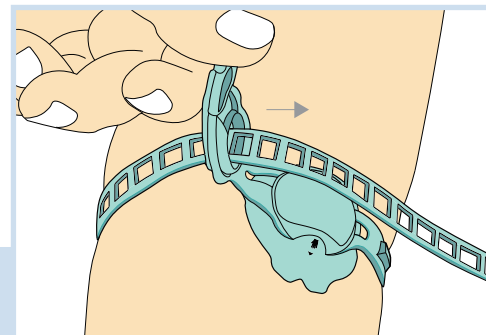

Release is single handed by lifting the buckle.

To complete the cannulation procedure, use a dressing to protect the IV insertion site as per standard practice.